

Take-Along Lifts LLC

Model 1000 Portable Transfer Aid

OWNERS INSTRUCTION MANUAL



DESCRIPTION

The Take-Along Lifts LLC Model 1000 is a portable patient transfer aid, designed to assist in transferring individuals weighing up to 300 pounds between surfaces such as beds, wheelchairs and toilets. It folds up compactly for storage and transport and weighs only 45 pounds, allowing it to be transported in a car, or rolled along like a piece of luggage.

INTENDED USE / INDICATIONS

The Model 1000 is designed to assist individuals weighing up to 300 lbs. who have sufficient upper body balance to support themselves in a sitting position on the edge of a bed. It is indicated for use in transferring these individuals between surfaces in close proximity to each other, and over flat, smooth surfaces. The transfer device is intended for use with standard-sized wheelchairs of approximately 18-inch frame size. The clearance between the legs of the lift is 24" at the base of the legs to 34" at the tips.

CONTRAINDICATIONS

The Model 1000 should not be used with individuals who do not have sufficient upper body balance to sit upright in a chair or on the edge of a bed. The PTA's slings are not intended to provide full body and head support. The Model 1000 should only be used in transferring individuals short distances, and should not be used over irregular or inclined surfaces, over obstacles, or on soft surfaces such as plush carpeting. The Model 1000 should not be used with individuals weighing more than 300 lbs. The Model 1000 may be used with "standard" sized wheelchairs which can fit between the legs of the lift. The clearance between the legs of the lift is 24" at the base of the legs to 34" at the tips.

SPECIAL NOTES

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS; CONTACT A QUALIFIED TAKE-ALONG LIFTS LLC REPRESENTATIVE BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

MAINTENANCE

Maintenance MUST be performed ONLY by qualified personnel.

WARNING notices as used in this manual apply to hazards or unsafe practices that could result in serious bodily harm.

CAUTION notices as used in this manual apply to hazards or unsafe practices which could result in minor personal injury or property damage.

NOTES highlight procedures and contain information which assist the operator in understanding the information contained in this manual.

NOTICE

The information contained in this manual is subject to change without notice.

SAVE THESE INSTRUCTIONS AND KEEP WITH PATIENT TRANSFER AID AT ALL TIMES.